
QuickConfig - Connecting to wheelchair for programming




Note: Wi-Fi required for download of app and initial account set-up

Getting started


QuickConfig wireless programming is compatible with Permobil Power Platform equipped wheelchairs.

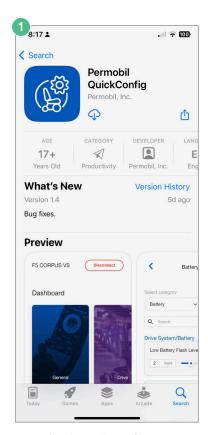

Download QuickConfig on your Android or Apple phone or tablet

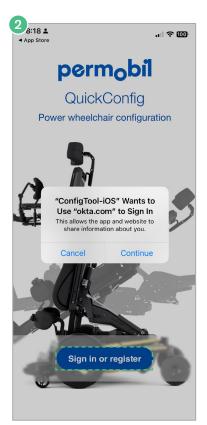

Tap "Sign in or register", then on pop-up screen tap "Continue" to be directed to single sign-on portal

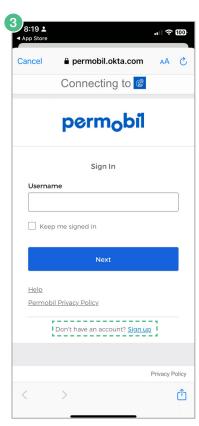


Tap "Sign up" to create your account



Enter your information to create account, using work email address

If prompted, contact your local Permobil representative to complete the registration process

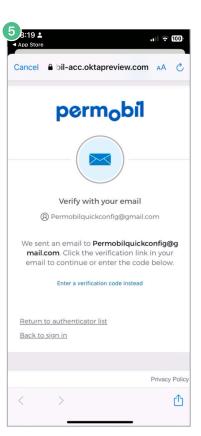

Check your email to verify account within 5 minutes


QuickConfig - Connecting to wheelchair for programming




Note: Wi-Fi required for download of app and initial account set-up

Getting started continued

QuickConfig wireless programming is compatible with Permobil Power Platform equipped wheelchairs.

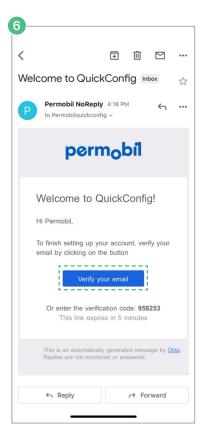



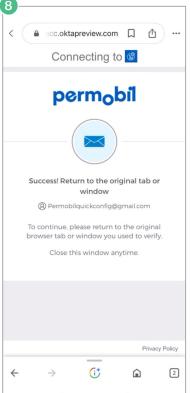



8:23 🕹

When prompted, return to the QuickConfig app

Read/accept QuickConfig end user license agreement



For more information on QuickConfig, scan the QR code or visit hub.permobil.com/en-au/quickconfig

Tap "Yes, It's me"

Tap "Verify your account" -OR-

Return to app and enter verification code


QuickConfig - Connecting to wheelchair for programming



Note: Wi-Fi required for download of app and initial account set-up

First time tutorial

3 di241 all → 100 di241 all → 100 di241 all → 100 di241

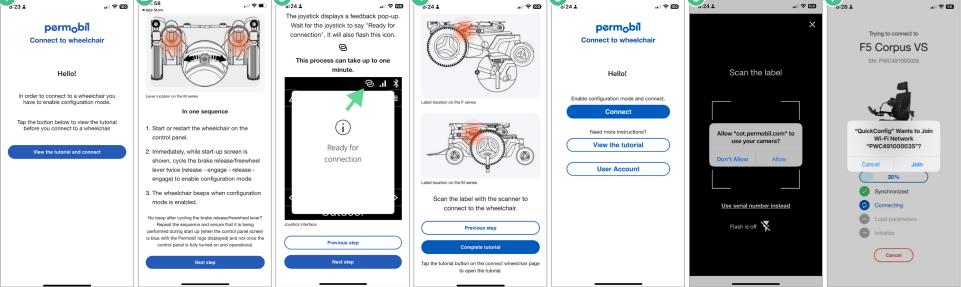

Tap "View the tutorial and connect"

Follow tutorial instructions to enable configuration mode on wheelchair using freewheel levers Wait until "Ready for connection" is present on input device Tap "Complete tutorial" to connect

Tap "Connect"

When requested, allow app to use camera, scan QR code on base to connect

-OR-

Connecting to wheelchair

Tap "Use serial number instead" to enter wheelchair serial number Tap "Join" to accept wheelchair's Wi-Fi network

Wait for completion of connection

You are ready to program