

Bluetooth™ mouse emulation for PC, Mac, & Android phone or tablet



Activating Bluetooth mode

1. From drive mode, press and hold soft key #3 (top right blue button on OMNI2 display) to enter settings menu.
2. Using navigation buttons on display, scroll down to “Bluetooth”, right navigation button to select.
3. Scroll to desired Bluetooth device (“Mouse 1 or “Mouse 2”), right or left command to turn <on>.
4. Scroll to “Exit” at bottom of screen, right command back to settings menu.
5. Scroll to “Exit” at bottom of settings menu, right command to “Exit”.
6. Power cycle wheelchair, wait approximately 8 seconds, power cycle again.

Pairing OMNI2 with device

1. Use mode button on OMNI2 display to enter Bluetooth Mode. Option to access via User Menu.
2. If more than one device paired, scroll to desired device and select with right command of input device plugged into OMNI2.
3. Put in discoverable mode by activating forward command of input device for 10 seconds or until you hear a beep, repeat by activating reverse command for 10 seconds or until you hear a second beep.
4. Enter Bluetooth settings on device being paired, and search for new device. Pair with “Mouse 1” or “Mouse 2”.



To click in “Mouse Mode”

Nudge or quick activation of driver control

Alternative Options:

- Consider use of “dwell” feature on computer or device
- If a separate external switch is required to click, utilize external Rnet Bluetooth Mouse Module or if PJS on chair, may program external switch on PJS as mouse click via Rnet PC Programming, Assignable Buttons.

How nudges work

- Left nudge - Left click
- Right nudge - Right click
- Forward nudge - Scroll up
- Reverse nudge - Scroll down

Auditory “beep” occurs when nudge command successfully completed.

Nudge time and actions, bluetooth screen graphic, and names of Bluetooth devices can be customized via Rnet PC Programming