




ROHO® Hybrid Select Cushion—Quick Setup Guide

  **Disclaimer:** This Setup Guide is not a substitute for the operation manual. The full manual, including safety information, must be reviewed prior to setting up the ROHO Hybrid Select Cushion.

Remove the cover and unlock the ISOFLO®

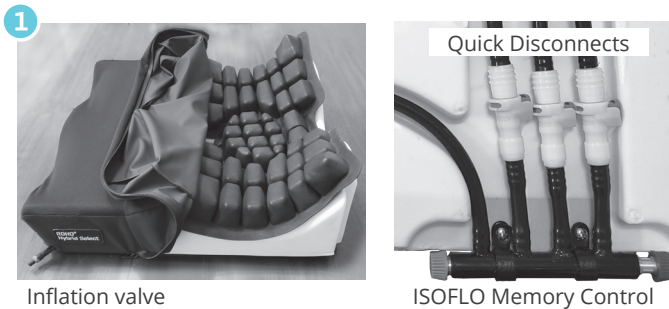

Using a stable surface, such as a table, unzip and open the cover. Turn the product over and make sure the quick disconnects are securely connected to the ISOFL0.


Note: When attaching the quick disconnects: The tubes can be attached in any order. If using the Hybrid Select without the IT Air Pad, refer to the "Offloading Option" in the provided instructions.



Make sure the ISOFL0 is unlocked. Refer to the operation manual.

Note: When the ISOFL0 is unlocked, the air flows freely through all of the air cells. You will not be able to inflate the cushion unless the ISOFL0 is unlocked.

Perform Atmospheric Inflation

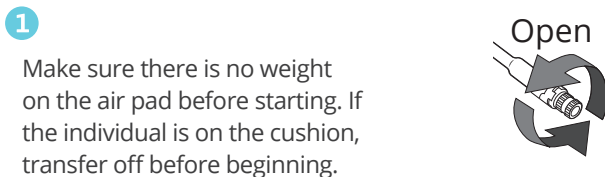

Make sure there is no weight on the air pad before starting. If the individual is on the cushion, transfer off before beginning.

2

Open the inflation valve.

Notes: To open, turn at least one full rotation. The inflation valve will spin freely when open. You can use the hand pump nozzle to open the inflation valve.

3

Inflate with the hand pump until all of the air cells feel firm.

Note: All air cells will inflate at the same time.




4

Remove the hand pump, and DO NOT close the inflation valve.



5

Allow air to escape from the inflation valve. Listen (or place a hand in front of the inflation valve) to see if the airflow has stopped.

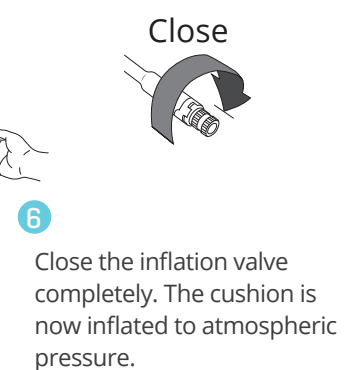

6

Close the inflation valve completely. The cushion is now inflated to atmospheric pressure.